

# Der mavista.com Newsletter vol. #7

## Introducing the **Jet-M System**

Exfoliate - Deeply Infuse - Hydrate - No-Needle Injection

Www.dermavista.com

800-767-6765

### In This Issue

## Jet-M System

See the Results

Revolutionary  
Technology

Jet-M Features


The Science

Seborrheic Keratosis

Der mavista is pleased to introduce the **Jet-M System**, a revolutionary device that delivers a supersonic jet stream of air and water to exfoliate, hydrate and infuse product into the dermis of the skin.

Sincerely,

Tom Jackson  
tom@der mavista.com  
1-800-767-6765

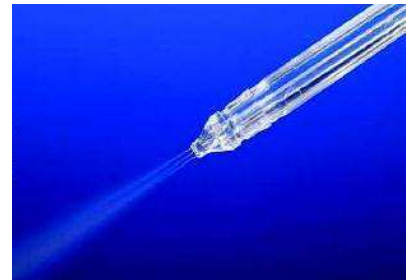

## See the Results

Upper Lip Wrinkles  
After 3 treatments



Crows Feet  
65 year old woman after 1 treatment




### For More Information

Visit  
[www.dermavista.com](http://www.dermavista.com)

## Der mavista .com


3172 Bradford Place  
Birmingham, AL 35242  
Voice: 800-767-6765  
Fax: 205-991-5396  
Email:  
info@der mavista.com

## Revolutionary Technology

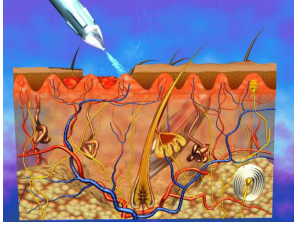


The **Jet-M System** custom handpiece creates a supersonic two-phase jet consisting of microdroplets of air and water. The pressure accelerates the droplets velocity to 200m per

second. When the microsized, fast-moving droplets form, the jet stream strikes the skin, and the kinetic energy of the droplets peels off the outer layers of dead skin.

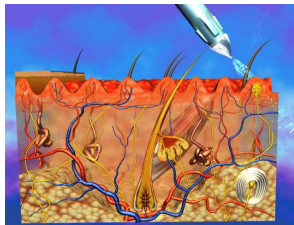

The pressure differential created by the jet stream opens up micro channels to dramatically enhance the infusion of skin rejuvenating molecules into the dermis.

During treatment skin temperature drops by 5-8 degrees Celsius. This, in turn, stimulates the body's thermoregulation system, increases blood circulation to the skin, and increases calorie consumption for several minutes.

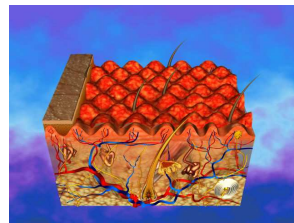

**Jet-M's** ability to immediately improve skin turgor, circulation, and humidity creates the ideal conditions for further healing and refines the skin structure. Post-operative use of **Jet-M** accelerates skin

reconstruction through deep and intensive moisturizing and cleansing, thereby stimulating the recovery process.

View a video of this process at [www.dermavista.com](http://www.dermavista.com).

## Jet-M Features

- Painless peeling
- Immediate and long-term wrinkle reduction
- Widens micro channels momentarily to enhance infusion
- Fights acne naturally without chemicals
- Instantly delivers soft, hydrated and supple skin
- Ability to perfuse skin supplements transutaneously without needles
- Produces no skin shedding

Jet-M can be used on any skin surface of the body and face, including the top of the head, total eye area and hands and feet.



View a video demo at [www.dermavista.com](http://www.dermavista.com)

## The Science

In a published dermatologist study, after two months of treatment 25 anti-aging patients reported *"expressed correction of mimic and age related wrinkles, as well as an average of 48% decrease in depth and width of wrinkles."*



Contact us at 1-800-767-6765 for an information booklet.

## Seborrheic Keratosis



### Get the latest info!

Sign up for our **email newsletters**.  
Click **Newsletters** at [www.dermavista.com](http://www.dermavista.com)

Dermavista.com is a subsidiary of Market Technologies, Inc. serving the aesthetic industry as distributor of Jet-M Systems, Dermatouch microdermabrasion systems, Vascutouch electro coagulation system, in addition to the Atache line of skincare products.